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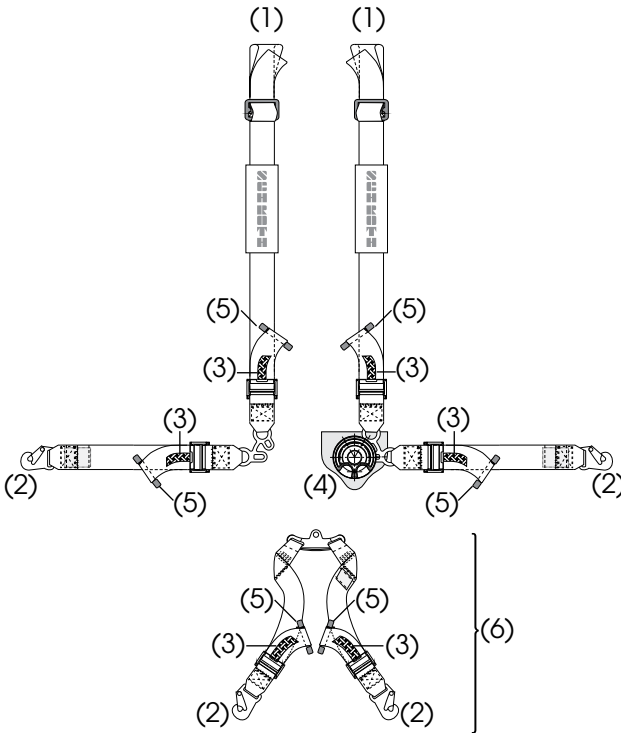
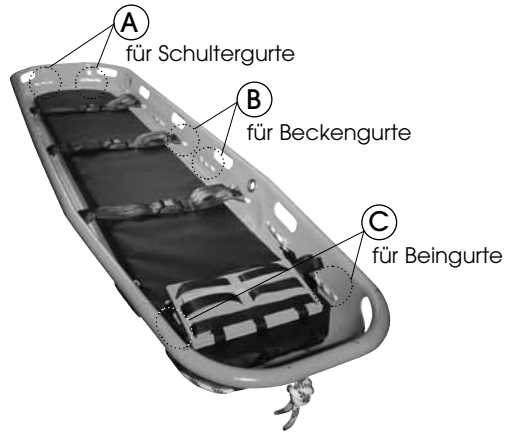
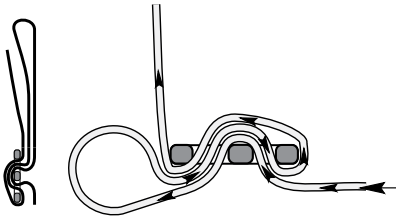
General Information		
1.1	Manufacturer	21
1.2	Customer Service Points	21
1.3	Declaration of Conformity	21
1.4	Warranty	21
1.5	Structure of Operating Manual	22
1.5.1	Purpose and Content	22
1.5.2	Storage	22
1.5.3	Used Symbols	22
2. Equipment Description		
2.1	Application Areas	23
2.2	Main Components	23
2.3	Technical Data and Dimensions	23
2.4	Storage and Use	24
3. Safety		
3.1	General Instructions	24
3.2	Counter Indications and Side Effects	25
3.3	User Requirements	25
3.4	Signs	25
4. Assembly		
4.1	Transport and Storage	26
4.2	Preparations	26
5. Use		
5.1	Equipment Use	28
5.1.1	How to Position the Injured Person on the Basket Stretcher	28
5.1.2	Operating Manual	28
5.1.2.1	How to Lift the Basket Stretcher	28
5.1.2.2	How to Load the Basket Stretcher into the Ambulance/ an Appropriate Vehicle	28
5.1.2.3	How to Unload the Basket Stretcher from the Ambulance/ an Appropriate Vehicle	28
5.1.2.4	Attaching the Footrest	29
5.1.2.5	Transporting the Basket Stretcher over Uneven Surfaces or Gradients	29
5.1.2.6	Vertical Transport	29
5.2	End of Use	30
5.3	Troubleshooting	30
6. Servicing		
6.1	Cleaning	31
6.2	Servicing	31
6.2.1	Important Check prior to each Use	31
6.2.2	Inspection Instruction	31
6.3	Customer Service	33
6.4	Life cycle	33
6.5	Inquiry Disposal	33
7. Content of Packaging Unit		34
Appendix I	How to Split and Assemble the 2-Part SAN-0087-2 Stretcher	35
Appendix II	Instructions for the Optional ultrABELT X-TREME Belt System	36
	Identification and Warranty Certificate/Record Card	38

1 - GENERAL INFORMATION

1.1 MANUFACTURER

ultraMEDIC is a qualified manufacturer of ambulance service equipment and a recognized specialist for rescue and emergency equipment.

Extensive expertise and cutting-edge technologies which are, apart from the manufacturing process and marketing phase, based on long-term research findings, are the best guarantee to assure a high level of customer satisfaction.

1.2 CUSTOMER SERVICE POINTS

Please contact the manufacturer or your dealer if there are any problems with using or servicing the equipment. Please always indicate the serial number for identification of the basket stretcher.

See Section 3.4

1.3 DECLARATION OF CONFORMITY

The basket stretchers have been manufactured in conformance with the applicable EC Directives. A declaration of conformity for this product can be obtained from the dealer or the manufacturer.

1.4 WARRANTY

The manufacturer offers a warranty of 24 months on this product. This warranty covers only the costs for replacements parts. Costs for labor or shipping are not included.

This warranty excludes damage as a result of:

- Transport and/or movement
- Handling errors
- Wrong, improper or no servicing of the basket stretcher
- Damage and/or breakage which can be attributed to faulty use of the basket stretcher
- Breakage of stretcher
- Normal wear
- Damage of components which solely serve an aesthetic function
- Repairs performed by unauthorized personnel
- Failure to use the product in conformance with the operation instructions
- Damage as a result of product use in inappropriate ambient conditions or under unsuitable circumstances

1.5 STRUCTURE OF OPERATING MANUAL

Please carefully read the operating manual. Proper preparation and correct use of the product are tantamount to the successful deployment of rescue equipment.

1.5.1 Purpose and Content

The purpose of this operating manual is to provide the customer with all the important information necessary for safe and effective use of the stretcher. This manual comprises information on technical data, use, servicing work, replacement parts and safety.

Carefully read the operating manual.

Please request the necessary information from the manufacturer if you are uncertain about how to interpret these instructions for use.

1.5.2 Storage

Keep this operating manual near the product. Keep this operating manual away from fluids and any other medium which may adversely affect the legibility of the manual.

1.5.3 Used Symbols


SYMBOL	MEANING	EXPLANATION
	DANGER	Indicates a hazard which may even be fatal to the person using the equipment.
	INSTRUCTIONS	Points out instructions or information on important functions or other useful information. It is imperative to carefully read and follow information with this symbol in the manual.
		The user is required CHECK to check all equipment parts prior to each use.
	LOOK UP	Read the operating manual.

2 - EQUIPMENT DESCRIPTION

2.1 APPLICATION AREAS

The basket stretcher/rescue basket is used for transporting injured persons. It can be used for any rescue situation and to handle difficult situations, where it is not possible to use regular equipment. No action of the injured person is required.

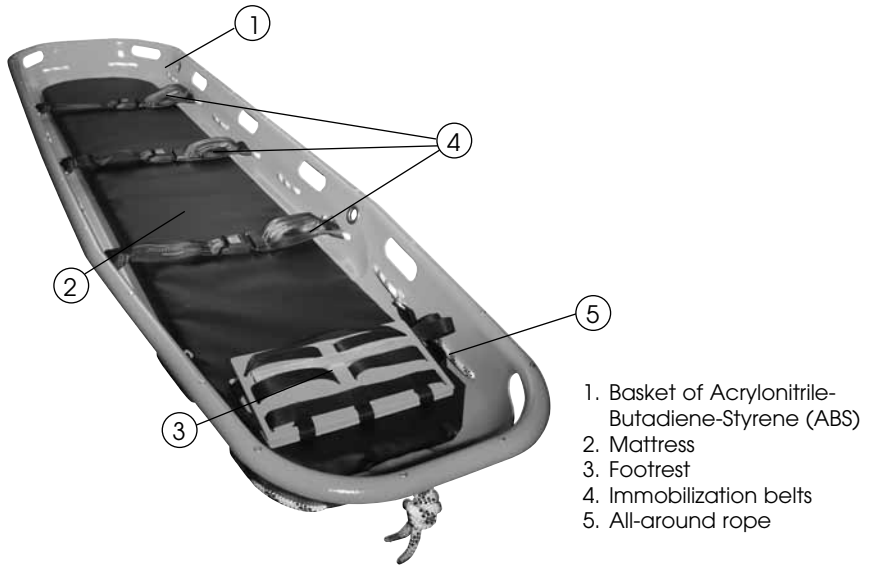2.2 MAIN COMPONENTS

2.3 TECHNICAL DATA AND DIMENSIONS

	ultraBASKET STRECHER 1-part SAN-0087-1	ultraBASKET STRECHER 2-part SAN-0087-2	ultraBASKET STRECHER SMALL SAN-0087-1M
Length	approx. 2,178 mm	approx. 2,141 mm approx. 1,150 mm, split	approx. 2,180 mm
Width	approx. 618 mm	approx. 618 mm	approx. 590 mm
Height	approx. 189 mm	approx. 189 mm	approx. 200 mm
Weight	8.5 kg	12 kg	12 kg
Load capacity	315 kg	315 kg	272 kg
Handles	12	12	12
Surface	ABS	ABS	ABS

2.4 STORAGE AND USE

Use and storage:

store and use at temperatures ranging from -20° to +60°C

3 - SAFETY

3.1 GENERAL INSTRUCTIONS

- Comments, instructions and warnings that point out potential risks are indicated by this symbol: 

The person who uses the equipment must carefully read the operating manual and in particular the safety measures and methods of application.

- Only an appropriately trained person should use this product. Please read the operating manual carefully and thoroughly to familiarize yourself with the use of this product. Practice using the empty basket stretcher prior to its first actual use.
- The use of the basket stretcher requires at least two persons who must be physically capable to carry out the task.
- Every time the basket stretcher is used, the injured patient must be immobilized in the basket stretcher; failure to ensure that the patient is properly and securely held in the basket stretcher may result in severe injury to the patient and/or the rescue worker.
- Do not exceed the maximum load capacity of the basket stretcher, indicated in the operating manual.
- Make sure the weight is evenly distributed.
- Regularly check the basket stretcher. Carry out the required servicing tasks, so that the basket stretcher is always ready for use. Maintenance at regular intervals increases the service life of the basket stretcher.
- Only use the basket stretcher as described in this operating manual. Do not under any circumstances arbitrarily modify the basket stretcher, so that it, for example, fits into a response vehicle or a rescue vehicle. This attempt to modify the basket stretcher may make the stretcher unusable and may even result in injuries to the patient or the rescue worker.
- Immediately take the basket stretcher out of service, should irregularities or problems occur, which may adversely affect the adequacy or the safety of the equipment.
- Do not, under any circumstances, leave the injured person unattended on the basket stretcher.

3.2 COUNTER INDICATIONS AND SIDE EFFECTS

There are no counter indications or side effects if these basket stretchers are used by qualified personnel familiar with first aid and rescue procedures.

3.3 USER REQUIREMENTS

ultraBASKET STRETCHERs are solely intended for professional applications. Rescue workers who intend to use these stretchers need to meet the following requirements. They must:

- be physically active/fit
- be able to firmly grab hold with both hands
- have a strong back and strong arms and legs
to be able to lift, push, and pull;
- have good muscle coordination.

Persons who are to use these stretchers must be instructed on how to efficiently, effectively and safely transport casualties.

This basket stretcher requires at least two persons who possess the physical strength, balance, coordination and common sense to complete the task.

Methods of how to properly lay an injured person, particularly a severely injured person on the basket stretcher and transport him/her across steep gradients or cope with extraordinary or unusual circumstances may require the deployment of several persons.

Before a person is assigned to a task involving this stretcher, his/her qualifications must be assessed.

3.4 SIGNS

The signs depicted below are fixed to the equipment and they must not be removed or covered.

In order to facilitate repair services, indicate the number (SN) on the sign every time.

ultraMEDIC	
<input type="checkbox"/>	ultraSCOOP STRETCHER SAN-0125 SAN-0288
<input type="checkbox"/>	ultraSPINE BOARD SAN-0280
<input type="checkbox"/>	ultraBASKET STRETCHER 1 piece SAN-0087-1 SAN-0087-1-M
<input type="checkbox"/>	ultraBASKET STRETCHER 2 pieces SAN-0087-2
Sea-Rescue	
<input type="checkbox"/>	ultraBASKET STRETCHER 1 piece (See-RC Zulassungsnummer: 438.003) SAN-0087-K
<input type="checkbox"/>	ultraBASKET STRETCHER 2 pieces (See-RC Zulassungsnummer: 438.004) SAN-0087-K-2
Serial No.: 26.001	
	2013 2014 2015
ultraMEDIC GmbH & Co. KG, 56564 Neuwied, Germany www.ultramedic.de	

4 - ASSEMBLY

4.1 TRANSPORT AND STORAGE

Make sure the stretcher is properly packaged prior to each use and try to avoid any impact or fall risks during transport.

Keep the original packaging in case you need to return the equipment for repair or servicing work. Damage to the basket stretcher as a result of use for transfer and movement is not covered under warranty. The customer is responsible for the costs of repair or compensation. The basket stretcher must be stored in a cool and dry place.

Do not deposit heavy material on the basket stretcher during storage; the basket stretcher is not to be used as storage space!

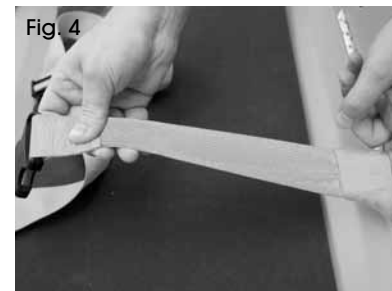
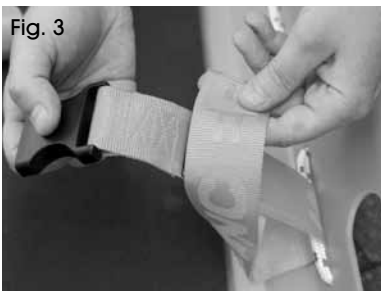
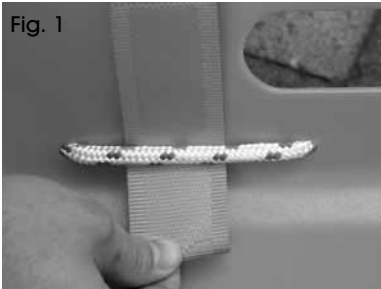
4.2 PREPARATIONS

On receipt of this product:

- remove the packaging and lay out the supplied materials;
- check these parts against the attached list.

In order to fit the safety belts, attach these to the all-around rope as shown in the following illustrations.

Inform the shipping agent immediately on receipt of the shipment of any damage to the cardboard box.

Then please also contact

ultraMEDIC GmbH & Co. KG
Im Bruch 13
D-56567 Neuwied, Germany
Tel. +49 (0) 263-196-9830
www.ultramedic.de
info@ultramedic.de

Check the following items, prior to using the basket stretcher:

- Are belts available to restrain/immobilize the injured person?
- Is the footrest included?
- Is the all-around rope pulled taut?

Only use the basket stretcher, if everything is in order.

Please contact the ultraMEDIC customer service department either at the address listed below or call the above-mentioned telephone number if you have any questions regarding the aforementioned check procedure.

These preparatory tasks must be completed prior to each use. Should the basket stretcher not be ready for correct and safe use, it must immediately be taken out of service and may only be used again after repair or after the elimination of damage.

5 - USE

Please observe the special features when using the 2-part basket stretcher.
See Appendix I, Page 33 for instructions

5.1 EQUIPMENT USE

Carefully read the operating manual prior to each use and practice using the basket stretcher with an empty basket stretcher.

5.1.1 How to Position the Injured Person on the Basket Stretcher

1. Before the injured person can be moved, lifted or transported, first, initial medical decisions will have to be made. After the diagnosis, inform the injured person of the possible existing risks and ask him/her (if possible) for assistance in positioning him/her on the basket stretcher.
2. Place the basket stretcher in a favorable position near the injured person prior to accommodating the patient on the basket stretcher.
3. Then transfer the injured patient from the previous immobilization system (spinal board vacuum mattress) to the basket stretcher.

5.1.2 Operating Manual

5.1.2.1 How to Lift the Basket Stretcher

1. Use the equipment belts to strap the injured person on the stretcher
2. Check to make sure that the belts are not too tight
3. One of the two rescue workers who are to use the basket stretcher stands at the foot end and the other at the head of the injured person.
4. In order to proceed effectively, the rescue workers take hold of the handles at each end of the basket stretcher. Only then is the basket stretcher to be lifted.

5.1.2.2 How to Load the Basket Stretcher into the Ambulance or another Appropriate Vehicle

1. Place the basket stretcher in the appropriate position.

5.1.2.3 How to Unload the Basket Stretcher from the Ambulance or another Appropriate Vehicle

1. Release the basket stretcher from all retention systems.
2. Take hold of the basket stretcher at the appropriate handles and pull it out of the vehicle.

5.1.2.4 Attaching the Footrest

1. Flip up the footrest and set it to a suitable height, so that the injured person cannot slide. Use the Velcro straps to fix the feet in place (see Fig.).

The injured person must be strapped in place on the basket stretcher with at least 3 belts.

If the person is injured at the lower limbs, then these will have to be restrained/immobilized by using appropriate equipment. Do not, under any circumstances, position the basket stretcher vertically in this case.

5.1.2.5 Transporting the Basket Stretcher over Uneven Surfaces or Gradients

1. Flip up the footrest and set it to a suitable height, so that the injured person cannot slide.
2. If the person is injured at the lower limbs, then these will have to be restrained/immobilized by using appropriate equipment. The injured person must be strapped in place on the basket stretcher with at least 3 belts.
3. Fasten the basket stretcher to a static safety system prior to raising or lowering it.

Attention: To this end please follow the instructions on how to fasten the transport rigging. Also read and follow the instructions for using the ultraBRIDLES transport rigging. Only use the basket stretcher with the ultraMEDIC ultraBRIDLES SAN-0086.

Check the condition of the injured person with each action you take.

5.1.2.6 Vertical Transport

1. Set the footrest to a suitable height, so that the injured person cannot slide.
2. If the person is injured at the lower limbs, then these will have to be restrained/immobilized by using appropriate equipment. The injured person must be strapped in place on the basket stretcher with at least 3 belts.
3. Fasten the basket stretcher to a static safety system prior to raising or lowering it.

Attention: To this end please follow the instructions for use of the transport rigging. Check the condition of the injured person with each action you take.

5.2 END OF USE

When the transport of the injured person has been completed, clean the basket stretcher as described in the Servicing section. Then store the basket stretcher.

5.3 TROUBLESHOOTING

PROBLEM	CAUSE	SOLUTION
The basket stretcher does not stay aligned when it is lifted.	Incorrect assembly of parts	Disassemble the parts and reassemble; then carefully check the basket stretcher.
Structural damage	Wrong application	Take the basket stretcher out of service and contact customer service

6 - SERVICING

6.1 CLEANING

Clean the disassembled parts with a soapy sponge; use a gentle cleaning agent. Use a soft cloth or buckskin to dry these parts. The surface material of the basket stretcher is made of very dense polyethylene (PE) (which can be easily cleaned) to facilitate cleaning and disinfection tasks. Water and soap can be used to clean the mattress and a regular disinfection agent can be applied for disinfection purposes.

Do not use a high-pressure/steam cleaner to clean the equipment! This may remove the lubrication and lead to corrosion.

Unless the basket stretcher is cleaned, a risk of infection exists caused by soiled surfaces of the basket stretcher.

6.2 SERVICING

If you properly service the basket stretcher at regular intervals, you will increase its service life. Determine a service schedule and the intervals for a regular check with the aid of a person responsible for the use of the basket stretcher, prior to its first use.

6.2.1 Important Check prior to each Use

- Serviceability of basket stretcher
- Condition of wear (belts, mattress)

6.2.2 Inspection Instruction

Important: : According to § 2 par. 2 MPBetreibV (operating regulation), rescue equipment "... may only be constructed, operated, applied, and maintained by persons holding the required qualification or knowledge and experience."

Inspection according to BGR 198/GUV-R 198

This inspection applies to the transportation suspension (ultraBRIDELS), the surrounding cord, and the associated patient fixation straps (see below).

UTo ensure the in-use safety of both the user and the injured person, we recommend to subject the following product to an annual inspection by an expert:

- ultraMEDIC basket stretchers SAN-0087-1 and SAN-0087-2 incl. the patient fixation strap, the surrounding cord, and the foot rest
- ultraMEDIC transportation suspension (ultraBRIDELS) SAN-0086 (expert rec

The following should be checked in the annual examination:

- Basket stretcher frame,
There must not be any sharp bends, deformations, cracks, or serious damage in general.
If damage is observed, the complete basket stretcher will have to be put out of use.

-Plastic tub,

Should not show any cracks, material fractures, or light-coloured rated break points.

If damage is observed, the complete basket stretcher will have to be put out of use.

-Surrounding cord,

Must fit tightly, must not show any cracks or damage.

The surrounding cord can be re-ordered from ultraMEDIC (or your dealer) and replaced.

-Patient fixation straps,

Must fit tightly, must not show any cracks or damage, and check that the push button locks work smoothly.

Patient fixation straps can be re-ordered from ultraMEDIC (or your dealer) and replaced.

-Foot rest,

Check belt straps for cracks or other damage, check hooks for deformations or cracks.

The aluminium profile of the foot rest sheets should fit tightly. Search for light-coloured rated break points and/or cracks.

Foot rests can be re-ordered from ultraMEDIC (or your dealer) and replaced.

Important information:

- Please do use ultraMEDIC spare parts only.
- Do not modify the equipment.
- The basket stretcher should be controlled regularly, even if there is no visible damage.

*

The following applies to the rope-down suspension:

Under normal conditions of use, a lifetime of 4-6 years may be assumed for joining means acc. to EN354 (see BGR/GUV-R 198).

The following applies to the ultraMEDIC / Skylotec rope-down suspension SAN-0086:

With proper storage and a storage period not exceeding 2 years, a maximum overall lifetime of 10 years beginning with the first usage will be possible, if the product belongs to a user who does not use it excessively, always uses it with care, and who knows the product's entire history of use. In the process, an examination by a qualified person should be carried out at least once a year, documenting the inspector's name and the detected characteristic features of the product.

Moreover, it must be made absolutely sure that the product has always been stored under optimum conditions, has never come in touch with chemicals, gases or other harmful substances, and that the total UV radiation over time has not exceeded the radiation occurring with intensive usage within a period of 4 years.

In any case, the manufacturer's operating instructions have to be observed.

6.3 CUSTOMER SERVICE INQUIRY

The customer may contact the manufacturer any time regarding information on use, servicing, assembly, etc. The customer should clearly phrase his/her questions and he/she should always refer to the operating manual.

6.4 LIFE CYCLE

The lifetime is depending on the individual operation conditions, whereas the plastic - ABS - Body are aging, regardless the care.

Product / Part	Lifetime
basket stretchers	12 years
surrounding cord	12 years
patient fixation straps	12 years
foot rest	12 years
transportation suspension (ultraBRIDELS)	6-10 years*

* see 6.2.2

6.5 DISPOSAL

Follow the applicable regulations.

7 - Contents of Packaging Unit

ultraBASKET STRETCHER 1-part Model: SAN-0087-1	ultraBASKET STRETCHER 2-part Model: SAN-0087-2	ultraBASKET STRETCHER extra narrow Model: SAN-0087-1M
1 rescue basket	1 rescue basket	1 rescue basket
1 footrest	1 footrest	1 footrest
1 mattress	1 mattress, split	1 mattress
3 immobilization belts	3 immobilization belts	3 immobilization belts
1 operating manual	1 operating manual	1 operating manual
	1 wrench (in belt)	

Optional accessories:

SAN-0084-G		Replacement belt/Immobilization belt
SAN-0086	ultraBRIDLE	Transport rigging suitable for all 3 models Length-adjustable transport rigging of robust nylon belt Strap for horizontal and vertical lifting of the rescue basket. Fastened with steel carabiner hooks
SAN-0088	ultraBASKET BAG	Stretcher bag and storage bag of manmade fiber fabric, suitable for SAN-0087-1, 1-part basket stretcher and SAN-0087-1F
SAN-0088-2	ultraBASKET BAG	Stretcher bag and storage bag of manmade fiber fabric, only for SAN 0087-2, 2-part basket stretcher
SAN-0260-65	ultraVAC 65	Vacuum mattress Made of heavy duty, flame-resistant fabric; X-ray permeable and heat-insulating. To facilitate the expedient and safe use, all vacuum mattresses are equipped with a snap-in valve and a vacuum limiter. Simply insert the upper section of the valve section and connect the hose to the pump. The valve is automatically sealed when the suction process is finished. It does not need to be closed. The snap-in valve is suita ble for CT applications and it is X-ray permeable. Dimensions: approx. 650 x 2000 mm; weight: approx. 4.5 kg
SAN-0278	ultraVACPUMP	Metal vacuum pump
SAN-0399	ultraPOLE	IV pole aluminum tripod, variable height adjustment, suitable for all stretchers and basket stretchers; can be screwed to a support rod, with spring clip and hook
SAN-0087-M		Replacement mattress, 1-part
SAN-0087-M		Replacement mattress, 2-part



APPENDIX I

These special features concern the ultraBASKET STRETCHER, 2-part (SAN-0087-2). Please observe.

HOW TO ASSEMBLE OR SPLIT THE 2-PART BASKET STRETCHER

1. Push together the stretcher
2. Lock the center lock (Fig. 1).
For this purpose a T-handle hex wrench is located in the belt (Fig. 2).
3. Lock both exterior quick-action locks (Fig. 3/4).

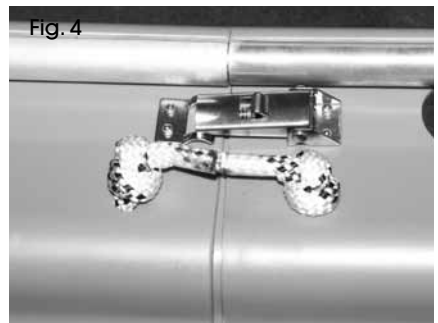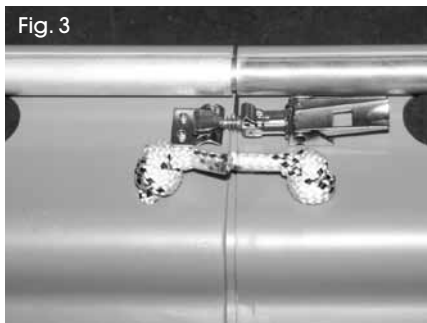Proceed in reverse order to disassemble/split the stretcher.

APPENDIX II

INSTRUCTIONS FOR THE OPTIONAL ultraBELT X-TREME BELT SYSTEM

The special 6-point ultraBELT X-TREME harness was specifically developed to quickly and securely hold patients in place. With ultraBELT X-TREME it is possible to quickly strap the injured person securely into the basket stretcher and handle even difficult transport situations.

HOW TO FIX THE BELT SYSTEM IN THE BASKET STRETCHER

Fasten the shoulder belts (1) to the two upper loops (A points) of the all-around rope. Loop the belts to fasten these as shown in the diagram.

The other two belts (2) which are attached to the shoulder belts can be used either as lap belts (fasten to all-around rope, B points) or as leg belts (fasten to all-around rope, C points).

If you use the belt as a *lap belt* it will be necessary to use an additional belt to fixate the feet. If you use the belt as a *leg belt* an additional lap belt is required.

The V-strap is fastened to the loops of the all-around rope.

HOW TO USE THE ultraBELT X-TREME BELT SYSTEM

First release all the belts using the belt releases (3) and then open the belt using the rotary buckle (4).

Put aside both shoulder belts with lap and leg belts and position the V-strap so that it points towards the foot end of the basket stretcher.

Lay the patient into the basket stretcher.

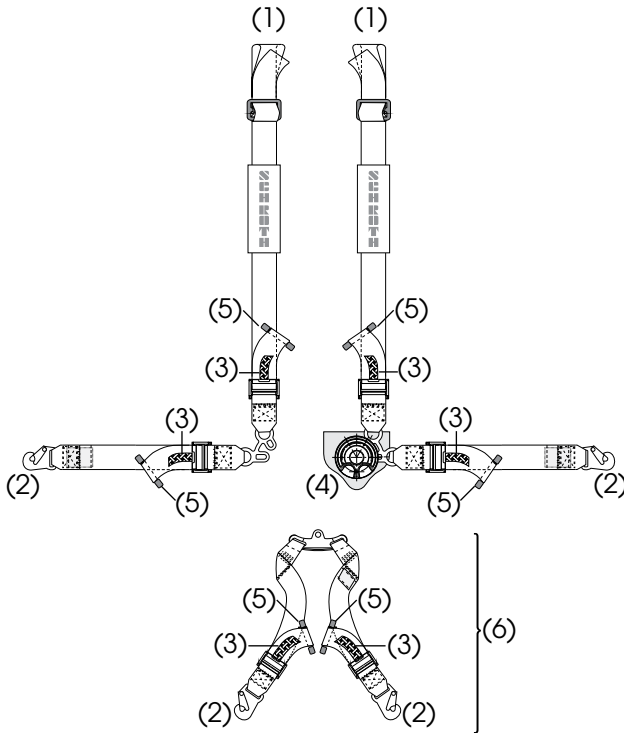
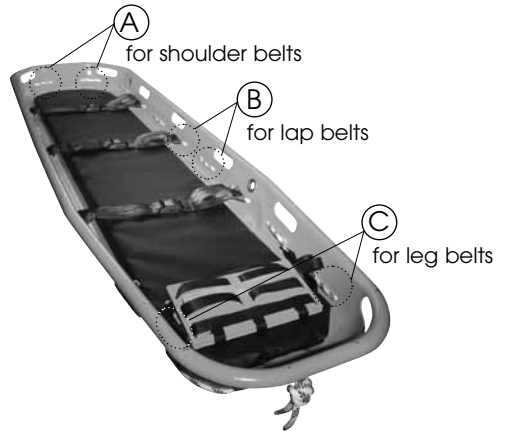
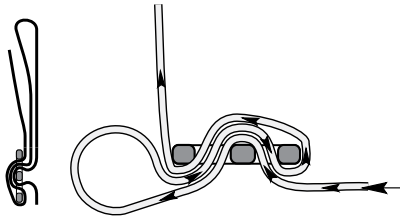
Place both shoulder belts (1) over the sides of the patient. The loose belt can now be fastened into the buckle of the other belt.

Guide the V-strap from underneath between the legs of the injured person and also buckle it.

Close the additional lap or leg belt (see above).

All belts can then be tightened with the belt tensioner (5).
The patient is now ready for transport.

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